


USE INSTRUCTIONS FOR ACTEEV PROTECT™ FABRIC MASKS WITH ANTIMICROBIAL PROPERTIES

Acteev
PROTECT™


HOW TO WEAR

1. Clean your hands with soap and water or alcohol-based sanitizer containing at least 60% alcohol.
2. Remove a mask from the box and make sure there are no obvious tears or holes on either side of the mask.
3. Hold the mask by the ear loops and identify the orientation of the mask.
4. Place the ear loops on your ears with the correct mask orientation.
5. Make sure the mask covers your nose, mouth and chin.
6. Clean your hands if your mask is being re-used.
7. Avoid touching your face even when the mask is in place.
8. If skin irritation occurs, immediately stop using the mask. If irritation persists, consult your primary care provider.

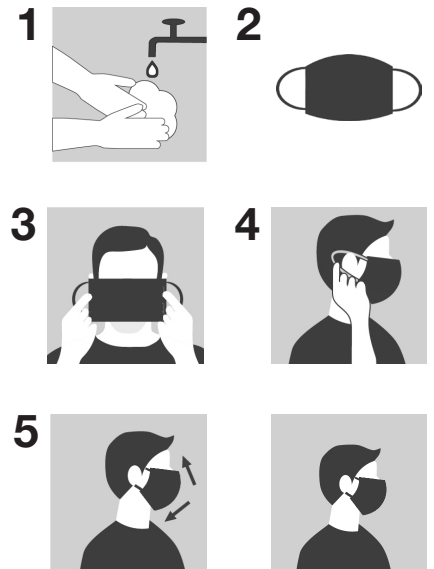

HOW TO REMOVE

1. Clean your hands before touching the mask.
2. Avoid touching the inside and/or the outside of the mask. They might be contaminated from your surroundings (outside) or from your coughing or sneezing (inside).
3. Hold both of the ear loops and gently lift and remove the mask.
 - Carefully fold inside out and place in a new plastic or paper bag until next use.
 - OR
 - Carefully fold inside out and follow the instructions for cleaning the masks.
4. Clean your hands again.

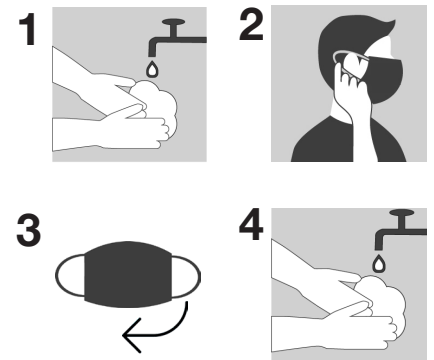

CARE INSTRUCTIONS

Always inspect the masks for damage. Discard if any puncture, holes, tears or significant wear is observed. Machine or hand wash with like colors.

Hand Wash:

1. Wash gently with soap under cool water, careful not to puncture fabric with nose bar.
2. Rinse thoroughly, squeeze gently and hang to dry.

Machine Wash:

1. Set washing machine to cold (~27 C or 80 F).
2. Placing masks in a laundry bag is recommended.
3. Use unscented detergents if you are sensitive to scents. Do not bleach.
4. Hang to air dry.
5. Store clean masks in a clean place when you are not using them.

For more instructions and best practices, refer to [cdc.gov](https://www.cdc.gov)



©2021 Ascend Performance Materials. The ASCEND PERFORMANCE MATERIALS and ACTEEV PROTECT marks and logos are trademarks of Ascend Performance Materials. Rev. 01/21

Although the information and recommendations set forth herein (hereinafter "Information") are presented in good faith and believed to be correct as of the date hereof, Ascend Performance Materials makes no representations or warranties as to the completeness or accuracy thereof. The full disclaimer of warranty and liability can be found at ascendmaterials.com/disclaimer.