

USE INSTRUCTIONS FOR ACTEEV PROTECT™ REUSABLE NONWOVEN PLEATED MASK

Acteev
PROTECT™


HOW TO WEAR

1. Clean your hands with soap and water or alcohol-based sanitizer containing at least 60% alcohol.
2. Remove a mask from the box and make sure there are no obvious tears or holes on either side of the mask.
3. Hold the mask by the ear loops & identify the orientation of the mask.
4. Place the ear loops on your ears with the correct mask orientation.
5. Make sure the mask covers your nose, mouth and chin.
6. Clean your hands if your mask is being reused.
7. Avoid touching your face even when the mask is in place.
8. If skin irritation occurs, immediately stop using the mask. If irritation persists, consult your primary care provider.



HOW TO REMOVE

1. Clean your hands before touching the mask.
2. Avoid touching the inside or outside of the mask. They might be contaminated from your surroundings (outside) or from your coughing or sneezing (inside).
3. Hold both ear loops and gently lift and remove the mask.
 - To re-use, carefully fold inside out and place in a new plastic or paper bag until next use.
 - If ready to dispose, just throw the mask in the trash.
 - If ready to wash, carefully fold inside out and follow the instructions for cleaning the masks.
4. Clean your hands again.

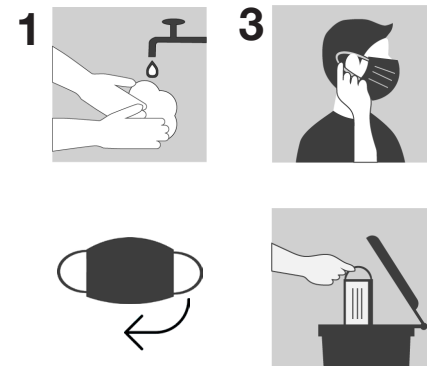

CARE INSTRUCTIONS

We suggest cleaning masks at regular intervals. They can be washed by hand up to 10 times.

1. Always inspect the masks for damage. Discard if any puncture, hole, tear or significant wear is observed.
2. Soak the mask in cold, soapy water for 5 min. Use unscented detergent if you are sensitive to scents.
3. Gently wash with squeezing motion for two minutes, taking care not to let the nose bar (if there is one) poke through.
4. Gently squeeze dry when done.
5. Rinse with fresh water 3 times or until water is clear. Make sure to gently squeeze dry every time.
6. Air dry the masks. Avoid drying in a dryer. Store clean masks in a clean place when you are not using them.

For more instructions and best practices, refer to [cdc.gov](https://www.cdc.gov)



©2020 Ascend Performance Materials. The ASCEND PERFORMANCE MATERIALS and ACTEEV PROTECT marks and logos are trademarks of Ascend Performance Materials. Rev. 8/20

Although the information and recommendations set forth herein (hereinafter “Information”) are presented in good faith and believed to be correct as of the date hereof, Ascend Performance Materials makes no representations or warranties as to the completeness or accuracy thereof. The full disclaimer of warranty and liability can be found at ascendmaterials.com/disclaimer.